# ENDO DONTICS

VOL.2

ENDODONTIE

ENDODONTICS

ENDODONTIE

ENDODONCIA

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## INHALT

CONTENTS  
CONTENU  
CONTENIDO

Seite Page Page Página		Seite Page Page Página	
4	<b>KOFFERDAM KLAMMERN</b> RUBBERDAM CLAMPS CRAMPONS À DIGUE ABRAZADERAS DE DIQUE DE GOMA	16	<b>MTA BLOCK UND SPATEL</b> <b>nach Dr. Edward S. Lee</b> MTA BLOCK AND DELIVERY CARVER acc. to Dr. Edward S. Lee BLOC DE MOULAGE MTA ET SPATULE suivant Dr. Edward S. Lee BLOQUE DE MTA Y ESPÁTULA según Dr. Edward S. Lee
6	<b>KOFFERDAM INSTRUMENTE</b> RUBBERDAM INSTRUMENTS INSTRUMENTS DIGUES INSTRUMENTOS PARA DIQUE DE GOMA	17	<b>WURZELKANAL-SPREIZER</b> ROOT CANAL SPREADERS FOULOIRS LATÉRAUX CONDENSADORES LATERALES
8	<b>ENDOBLACK® – Die schwarze Beschichtung</b> ENDOBLACK® – The black coating ENDOBLACK® – Le recouvrement noir ENDOBLACK® – El recubrimiento negro	17	<b>WURZELKANAL-STOPFER</b> ROOT CANAL PLUGGERS FOULOIRS À CANAUX OBTURADORES RADICULARES
10	<b>INSTRUMENTENSET 8305 FÜR DIE KONVENTIONELLE ENDOTHERAPIE</b> SET OF INSTRUMENTS 8305 FOR CONVENTIONAL ENDODONTICS SET D'INSTRUMENTS 8305 POUR L'ENDODONTIE CONVENTIONELLE KIT DE INSTRUMENTOS 8305 PARA LA ENDODONCIA CONVENCIONAL	17	<b>ENDODONTIE PINZETTE</b> ENDODONTIC FILE/POSTHOLDER PINCE ENDODONTIQUE PINZA DE ENDODONCIA
11	<b>INSTRUMENTENSET 8306 FÜR DIE ENDODONTOLOGISCHE CHIRURGIE</b> SET OF INSTRUMENTS 8306 FOR ENDODONTIC SURGERY SET D'INSTRUMENTS 8306 POUR LA CHIRURGIE ENDODONTIQUE KIT DE INSTRUMENTOS 8306 PARA LA CIRUGÍA ENDONTOLÓGICA	18	<b>MUNDSPIEGELGRIFFE UND MUNDSPIEGEL</b> MOUTH MIRROR HANDLES AND MOUTH MIRRORS MANCHES MIROIRS ET MIROIRS MANGOS PARA ESPEJOS Y ESPEJOS DE BOCA
12	<b>SONDEN</b> EXPLORERS SONDES EXPLORADORES	19	<b>MIKRO MUNDSPIEGEL</b> MICRO MOUTH MIRRORS MICRO MIROIRS À BOUCHE MICRO ESPEJOS DE BOCA
13	<b>DIVERSE INSTRUMENTE</b> VARIOUS INSTRUMENTS DIVERS COMPLÉMENTAIRES DIVERSOS INSTRUMENTOS	20	<b>GUTTAPERCHA ENTFERNER</b> <b>nach Dr. Yoshi Terauchi</b> GUTTAPERCHA REMOVAL INSTRUMENTS acc. to Dr. Yoshi Terauchi INSTRUMENTS POUR ENLEVER LA GUTTAPERCHA suivant Dr. Yoshi Terauchi INSTRUMENTO PARA SACAR LA GUTTAPERCHA según Dr. Yoshi Terauchi

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- 22** **SET ZUR NICHT-CHIRURGISCHEN REVISIONSBEHANDLUNG**  
**nach Dr. Antonis Chaniotis**  
NON-SURGICAL RETREATMENT KIT  
acc. to Dr. Antonis Chaniotis  
KIT DE RETRAITEMENT NON CHIRURGICAL  
suivant Dr. Antonis Chaniotis  
KIT DE RETRATAMIENTO NO QUIRÚRGICO  
según Dr. Antonis Chaniotis
- 24** **THE LA ENDO SURGICAL KIT – INSTRUMENTENSET FÜR DIE ENDO-CHIRURGIE**  
**nach Dr. David Han**  
THE LA ENDO SURGICAL KIT –  
INSTRUMENTS SET FOR ENDODONTIC SURGERY  
acc. to Dr. David Han  
THE LA ENDO SURGICAL KIT –  
SET D'INSTRUMENTS POUR LA CHIRURGIE  
ENDODONTIQUE suivant Dr. David Han  
THE LA ENDO SURGICAL KIT –  
KIT DE INSTRUMENTOS PARA LA CIRUGÍA  
ENDODONTOLÓGICA según Dr. David Han
- 26** **ENDO-CHIRURGIE RETRAKTOREN**  
**nach Dr. David Han**  
ENDODONTIC SURGERY RETRACTORS  
acc. to Dr. David Han  
ÉCARTEURS ENDODONTIQUES CHIRURGICALES  
suivant Dr. David Han  
RETRACTORES PARA LA ENDODONCIA  
QUIRÚRGICA según Dr. David Han
- 27** **ENDO-CHIRURGIE RETRAKTOREN**  
**nach Dr. Richard Rubinstein**  
ENDODONTIC SURGERY RETRACTORS  
acc. to Dr. Richard Rubinstein  
ÉCARTEURS ENDODONTIQUES CHIRURGICALES  
suivant Dr. Richard Rubinstein  
RETRACTORES PARA LA ENDODONCIA  
QUIRÚRGICA según Dr. Richard Rubinstein
- 28** **WUNDHAKEN**  
RETRACTORS  
ÉCARTEURS  
SEPARADORES

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- 29** **ENDO-CHIRURGIE RETRAKTOREN**  
**nach Prof. Syngcuk Kim**  
ENDODONTIC SURGERY RETRACTORS  
acc. to Prof. Syngcuk Kim  
ÉCARTEURS ENDODONTIQUES CHIRURGICALES  
suivant Prof. Syngcuk Kim  
RETRACTORES PARA LA ENDODONCIA QUIRÚRGICA  
según Prof. Syngcuk Kim
- 30** **PENN ENDO CREATION SETS**  
**nach Prof. Syngcuk Kim**  
PENN ENDO CREATION SETS  
acc. to Prof. Syngcuk Kim  
SETS PENN ENDO CREATION  
suivant Prof. Syngcuk Kim  
KIT PENN ENDO CREATION  
según Prof. Syngcuk Kim
- 36** **SKALPELLKLINGEN UND HALTER**  
SCALPEL BLADES AND HANDLES  
LAMES DE BISTOURI ET MANCHES  
HOJAS DE BISTURÍ Y MANGOS
- 37** **MINI UND MINI-MIKRO KLINGEN, biegsam**  
MINI AND MINI-MICRO BLADES, bendable  
LAMES BISTOURI MINI ET MINI-MICRO, pliables  
HOJAS DE BISTURÍ MINI Y MINI-MICRO, doblables
- 38** **MTA APPLIKATOREN**  
MTA CARRIERS  
PORTES-MTA  
PORTAS MTA
- 39** **ENDODONTIE-ZÄNGCHEN**  
ENDODONTIC FORCEPS  
PINCE ENDODONTIQUE  
PINZA DE ENDODONCIA
- 40** **MIKRO-NADELHALTER MIT HARTMETALL (TC) EINSÄTZEN**  
MICRO NEEDLE HOLDERS WITH  
TUNGSTEN CARBIDE INSERTS  
MICRO PORTE-AIGUILLES AVEC  
CARBURE DE TUNGSTÈNE  
MICRO PORTA AGUJAS CON  
CARBURO DE TUNGSTENO

KOFFERDAM KLAMMERN

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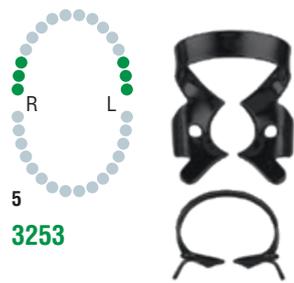
- black oxide coating to reduce eye fatigue by minimizing light reflection
- long product lifetime and consistent quality spring action
- available in various sizes and shapes
- excellent contrast and minimal light reflection in photographs

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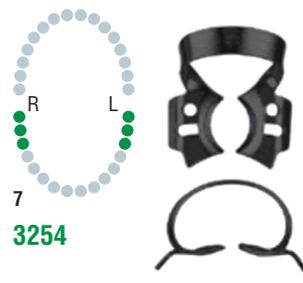
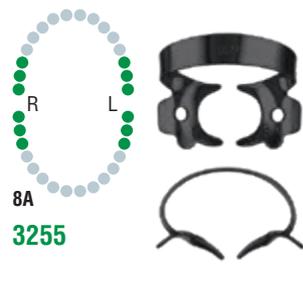
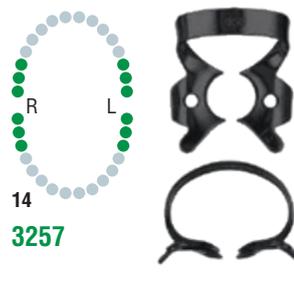
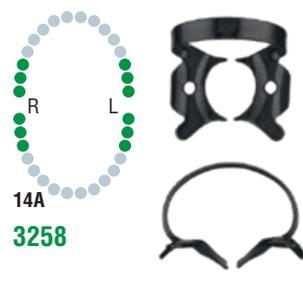
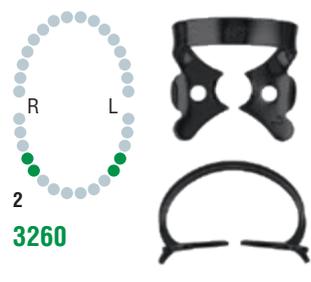
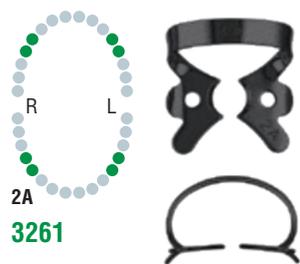
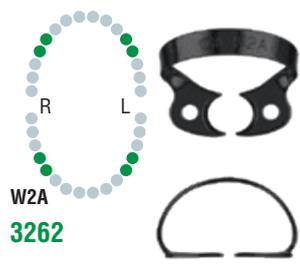
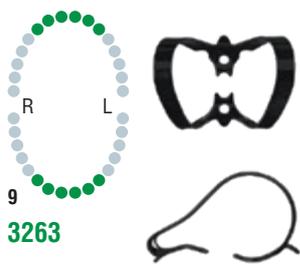
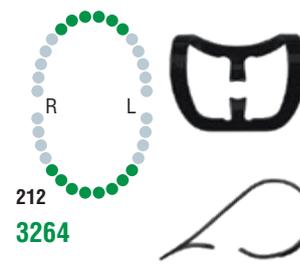
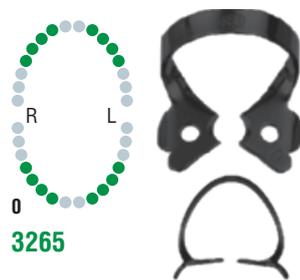
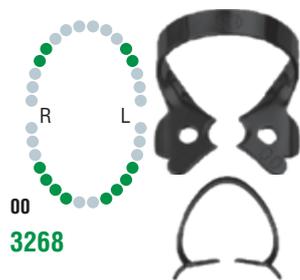
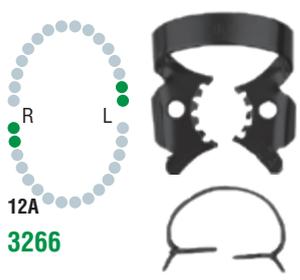
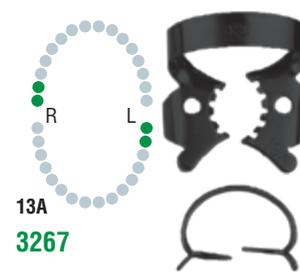
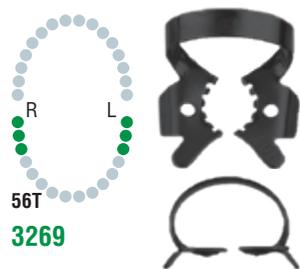
Rubberdam punch forceps AINSWORTH with silicone handle and 5 holes disc

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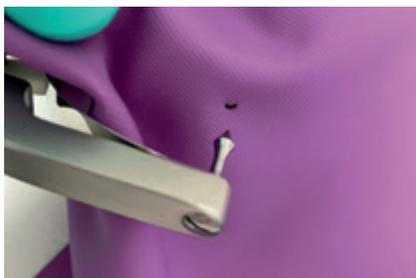
The ergonomic silicone handle offers a secure feel with pleasant grip characteristics.

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The antireflective coating makes it easy to work under the operating microscope light and more generally avoids visual fatigue during the different clinical stages when creating the access cavity for root canal obturation.

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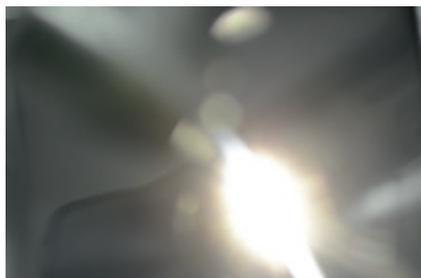
COMPARISON OF DIFFERENT SURFACE TREATMENTS UNDER DIRECT EXPOSURE OF LIGHT:

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
Mirror finish: extreme dazzling effect due to strong reflection

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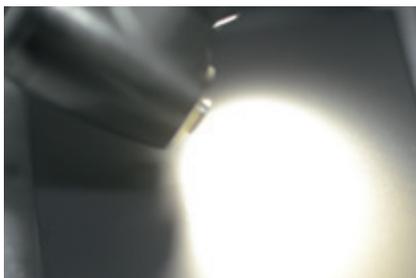Sandblasted finish: less dazzling, reduced reflection

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ENDOBLACK® coating: reflection is reduced considerably, dazzling effect is lowered significantly

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#### ENDOBLACK® – THE BLACK COATING OFFERS FURTHER ADVANTAGES:

- high hardness (approx. 10Mpa, corresponds to approx. 1000-1300HV)
- reduced friction (friction coefficient is in average 7 times lower compared with Titanium and Stainless Steel) thus reduced wear and tear and increased product lifetime
- excellent chemical properties: increased surface density, biocompatible barrier, insensitive with cleaning solutions, reduced risk of contact corrosion

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SET OF INSTRUMENTS 8305 FOR CONVENTIONAL ENDODONTICS

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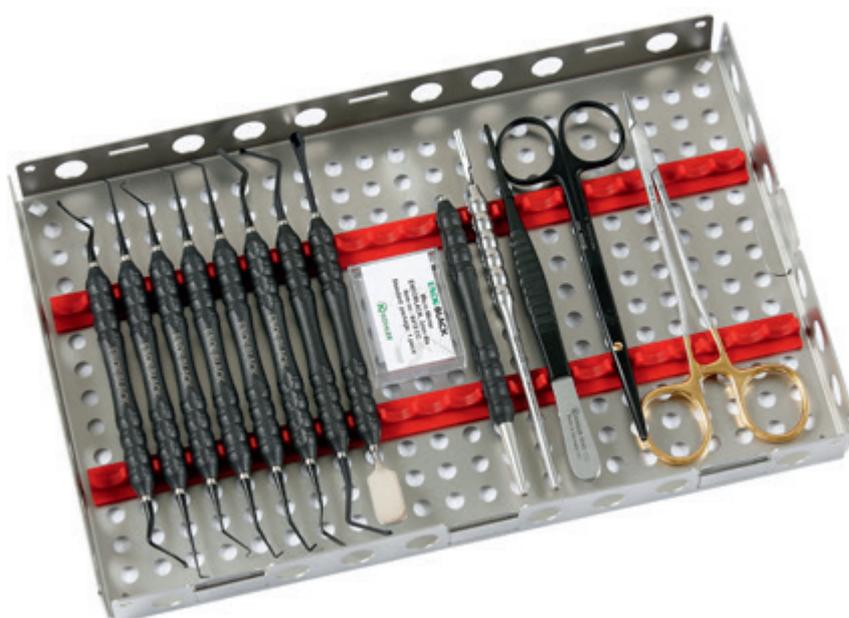
SET OF INSTRUMENTS 8306 FOR ENDODONTIC SURGERY

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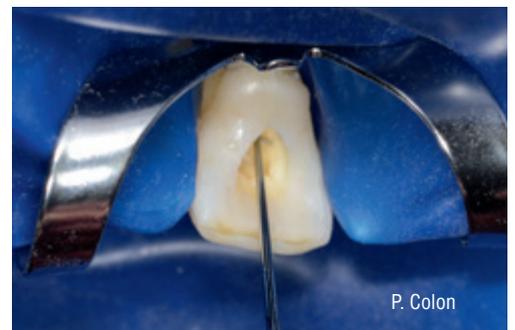
nach/acc. to/suivant/según

**Professor Pierre COLON**  
 (DDS, PhD, habilitation,  
 Head of Department  
 Operative Dentistry and  
 Endodontics,  
 Paris Diderot University)

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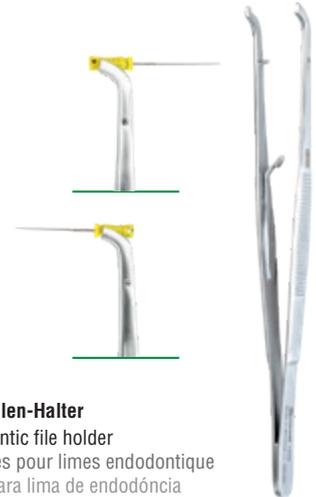
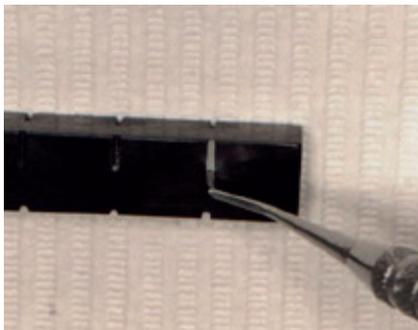
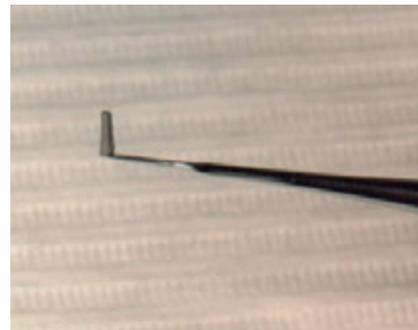
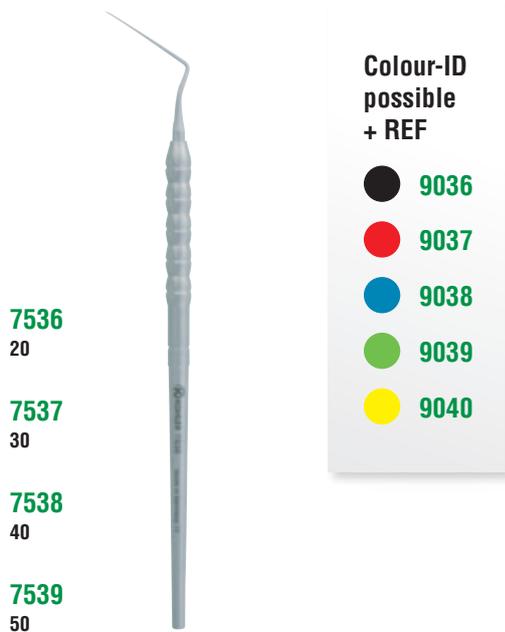
**Dr. Edward S. Lee** attended college at the University of California, San Diego. He received his DDS degree from the University of California, San Francisco and practiced general dentistry for several years before receiving his certificate in endodontics from the University of California, Los Angeles. Dr. Lee is a recognized innovator in the field of endodontics. He developed a new delivery technique for root fillings and holds a patent on this delivery device. He

has lectured nationally and internationally about this technique and has published articles in peer-reviewed journals. Dr. Lee’s current interest is in regenerative endodontics and the healing potential of Platelet-Rich Fibrin (PRF). His latest publication is “Platelet -Rich Fibrin in Endodontics.” Dr. Lee is currently teaching at the UCSF Post Graduate Endodontic clinic. Dr. Lee completed a Dental Humanitarian Project providing endodontic care to

orphans and pediatric patients in Suqian China. Dr. Lee is a member of the American Association of Endodontists, California State Association of Endodontists, Northern California Academy of Endodontics (past president), Pacific Endodontic Research Foundation, American Dental Association, California Dental Association, and the San Francisco Dental Society. Dr. Lee has served as a Board Director for the California Dentists Guild.

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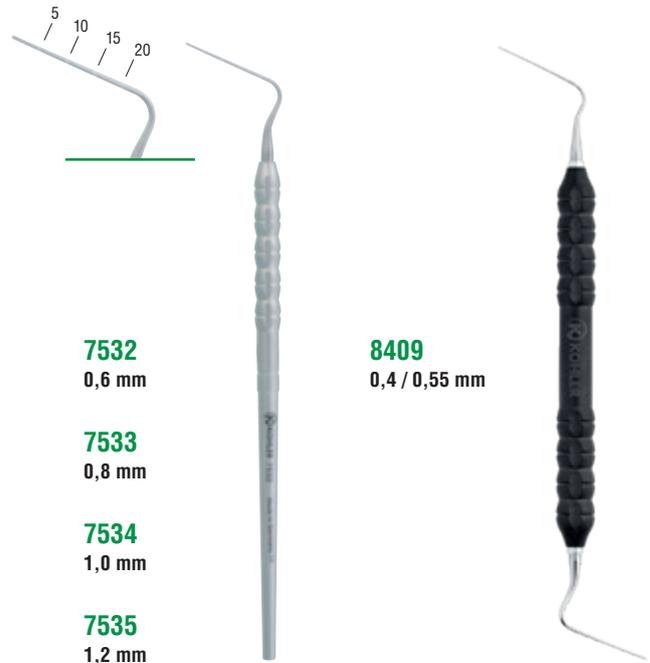
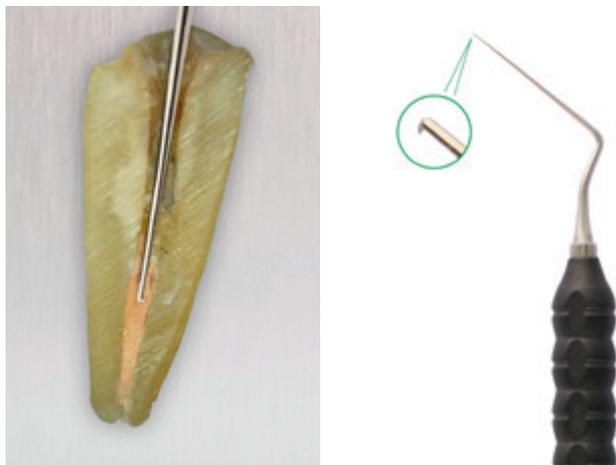
All mirrors and mirror handles have a European thread (simple stem), unless otherwise indicated.

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The instruments have been specially-designed for the orthograde revision of an endodontic treatment. The micro-hooks on the fine working ends are produced to utmost precision and ensure a tight hooking into the Guttapercha, at the same time they provide an outstanding tensile strength and stability.

The handle made of PEEK, a shape-and color-stable high-performance plastic, is extremely light – each instrument weighs only 12 grams. The handle design combines the advantages of simple cleaning and sterilization with outstanding grip characteristics.

The instruments are available in four versions, adapted to the different clinical situations: length of working ends 26 mm or 18 mm, hooks to left / right ◄► or up / down ◊.

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without hook, diamond coated tips, for Guttapercha removal and penetration of calcified or ledged canal

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**Yoshitsugu Terauchi D.D.S., PhD.**

Dr. Terauchi is a part-time lecturer at Tokyo Medical & Dental University and maintains a private practice limited to endodontics since 1998. He earned his DDS in 1993 and completed his residency at Tokyo Medical & Dental University in 1995, where he also received his PhD from the Department of Endo-

odontics. He has published several articles in peer-reviewed journals nationally and internationally. He also has authored several chapters in textbooks including the 11th edition of "Pathways of the Pulp". He has lectured nationally and internationally and has appeared twice on National TV, for modern endodontics.

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8251

Fotos Dr. Antonis Chaniotis

**Dr. Antonis Chaniotis DDS, MDSC**  
 Chaniotis Antonis is a graduate of the University of Athens Dental School, Greece (1998). In 2003 he completed the three-year postgraduate program in Endodontics at the University of Athens Dental School. Since 2003, he owns a limited to microscopic Endodontics private practice in Athens, Greece.  
 For 10 years, he served as a clinical instructor affiliated with the undergraduate and postgraduate programs at the Uni-

versity of Athens, Athens Dental School, Endodontic department, Greece.  
 In 2012 he was awarded the title of Clinical fellow teacher at the University of Warwick, Warwick dentistry UK.  
 He lectures nationally and internationally, and he has published articles in local and international Journals.  
 He currently serves as an active member of the Hellenic Association of Endodontists, Country representative in ESE, member of the Hellenic Society of

Endodontology, certified member of the European Society of Endodontology (ESE), international member of the American Association of Endodontists (AAE) and member of the European Academy of Digital Dentistry

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THE LA ENDO SURGICAL KIT – INSTRUMENTS SET FOR ENDODONTIC SURGERY ACC. TO DR. DAVID HAN

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Apicoectomy retractors, toothed  
The apicoectomy retractors assist to hold the gingival flap of the gingival during the endodontic surgery.

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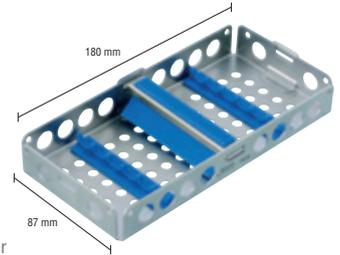
**Prof. Kim**

"The perio probes and explorers are important not only for checking perio probing and related measurements but additionally to check the height of tooth. By placing the explorer on the buccal surface of the tooth, one can estimate the height of the tooth where the osteotomy must be made"

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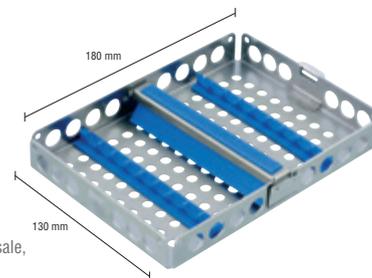
**Prof. Kim**  
 "The two periosteal elevators are important for flap elevation without tearing. The four ends have different shapes. The sharp end is the one which lifts tissue at the sulcus without tearing. The different shapes of the remaining tips are to accommodate the tissue type and thickness of

the tissue. These curettes are the most important instruments in the kit!! To remove all the granulation tissue embedded into the bone is the hardest job in the surgery! There are many shapes of curette tips; some are sharp and some are thick, dependent upon the area in which you are doing the surgery. Around the roots one needs sharp

curved curettes. Removing large granulation tissue requires a large curved curette. The osteotomy site shape varies - some are big and deep, and some are shallow and curved laterally. These various shaped curettes are designed to remove all granulation tissue efficiently"

PENN ENDO CREATION SETS NACH PROF. SYNGCUK KIM PLUGGER SET

PENN ENDO CREATION SETS ACC. TO PROF. SYNGCUK KIM  
 SETS PENN ENDO CREATION SUIVANT PROF. SYNGCUK KIM  
 KIT PENN ENDO CREATION SEGÚN PROF. SYNGCUK KIM

PLUGGER SET  
 SET DE FOULOIRS  
 KIT DE COMPACTADORES

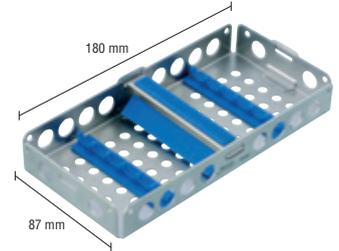
8309

**Plugger Set**  
 Plugger Set  
 Set de fouloirs  
 Kit de compactadores

7600-BL

**Mini Tray mit Quersteg, für max. 6 Instrumente**  
 Mini Tray with transverse bar, for up to 6 instruments  
 Mini Tray avec entretoise transversale, pour 6 instruments au maximum  
 Mini Tray con trabilla transversal, para un máx. de 6 instrumentos

8460

**MTA Spatel/ Träger**  
 MTA delivery/ carver instrument  
 Spatule/ porteur MTA  
 Espátula/ portador MTA

8461

**Instrument zur Reparatur von Perforationen**  
 Perforation repair instrument  
 Instrument pour réparer perforations  
 Instrumento de reparación de perforaciones

8462

**Mikro Plugger**  
 Micro plugger  
 Fouloir micro  
 Compactador micro

8420

**MTA Block**  
 MTA Block  
 Bloc MTA  
 Bloque MTA

8463

**Mikro Plugger**  
 Micro plugger  
 Fouloir micro  
 Compactador micro

**Prof. Kim**  
 "Plugging instruments: After placement of bioceramic materials or other filling materials, the root prepared canals must be gently condensed coronally.

The Plastic Instruments above place the filling material into the root prepared canal. There are two sizes of pluggers - thin and thick – which are used to plug the materials deeply into the canal. The size of the micro-condenser tips used depends upon the size of root end prep. The plastic instrument is designed to press the root end filling material into the root end prep space. The MTA Block is a special system which allows the dentist to pick small piece of MTA or bioceramic material from the plastic block with the plastic instrument and carry to the root end prep canal!"

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**8466**  
**4 mm**

**Mikro Spiegel**  
 Micro mirrors  
 Micro miroirs  
 Micro espejos

**Prof. Kim**  
 "There are three sizes of micro-mirrors with flexible shaft: 4 mm, 3 mm and 2 mm width in diamond shapes. Traditional round shaped

mirrors have limited usage for anterior large osteotomy cases. In most cases e.g., bicuspid or molar, the osteotomy size varies; some are small so 2 mm or 3 mm is needed. When the

osteotomy is large, 4 mm is needed. Since the resected root is elongated buccal and lingual, diamond shaped mirrors with proper width are optimal."

NAHTINSTRUMENTE

SUTURE INSTRUMENTS  
 INSTRUMENTS DE SUTURE  
 INSTRUMENTOS DE SUTURA

**7678-S**

**15 cm**  
**Mikro-Schere, gezahnt**  
 Micro scissors, serrated  
 Ciseaux micro, dentelés  
 Tijeras micro, dentadas

**7660**

**14 cm**  
**Mikro-Nadelhalter**  
 Micro needle holder  
 Micro porte-aiguilles  
 Micro porta agujas

**5603**

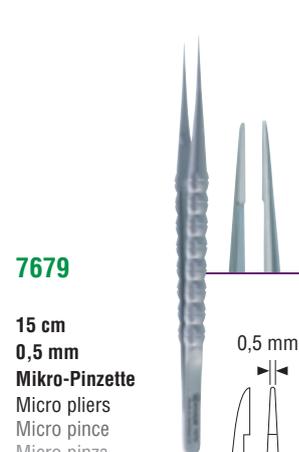
**8,5 cm**  
**0,5 mm**  
**Pinzette BISHOP-HARMAN**  
 Pliers BISHOP-HARMAN  
 Pince BISHOP-HARMAN  
 Pinza BISHOP-HARMAN

**7679**

**15 cm**  
**0,5 mm**  
**Mikro-Pinzette**  
 Micro pliers  
 Micro pince  
 Micro pinza

**Prof. Kim**  
 "These instruments are critical for suturing. The micro forceps stabilizes the tissue and the micro needle holder holds the suture, while the micro scissors cut

sutures microscopically. Regular large instruments cannot perform the task since 5.0 or 6.0 suture is really thin!"

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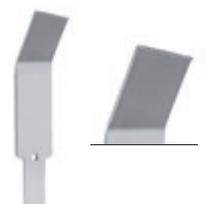
**Prof. Kim**  
 "These new retractors are specially-designed. The thin serrated blades efficiently hold the tissue atraumatically. The blades are angled

left and right 15 degrees. When doing surgery the patients' chins are NOT straight – they are always tilted to right or left slightly. To

compensate for this defect, the newly designed retractors with 15 degree angles will anchor the retractors at the correct level."

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**Prof. Kim**

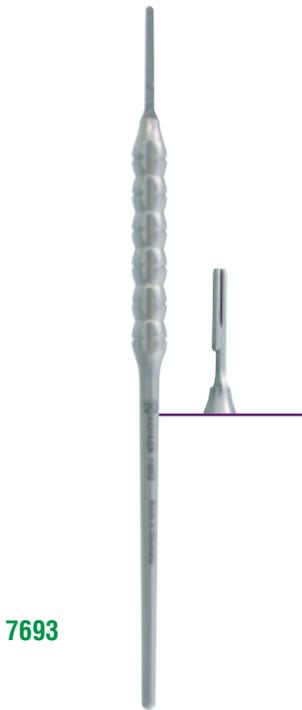
"These are two special instruments designed for the Bone Window Technique. The prepared bone window can be lifted easily with the specially-designed „thin“ blades made of extremely hard stainless steel. These are MUST instruments if one wants to perform the Window Technique."

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SCALPEL BLADES AND HANDLES  
LAMES DE BISTOURI ET MANCHES  
HOJAS DE BISTURÍ Y MANGOS

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All blades are bendable – except where indicated with \* – by means of a forceps or needle holder, up to max. 45 degrees

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- extra slim, serrated working part
- fine teeth and a longitudinal groove in the beak for a safe grip
- three-stage locking – once the ratchet is fixed, the beaks grasp even the thinnest diameters easily
- the inner spring system leaves the beak in a slightly open position – ready to use – and offers a sensitive handling

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Micro forceps with delicate, flat tips, diamond coated, also to securely hold parts, inlays, etc.

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The indication for suture sizes is only an approximate value. We assume no liability for their accuracy and final assignment concerning the loading capacity of the needle holders. The responsibility lies solely with the user. All TRINOVO® micro needle holders are available with black coating at an extra charge – for reduced reflection. Simply add the abbreviation „-CC“ to the item number of the needle holder, e.g. 7660-CC.

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**Kohdent Roland Kohler Medizintechnik GmbH & Co. KG**